


Saliva Collection Instructions

Do not eat, drink, chew gum, brush your teeth, use mouthwash, or smoke for at least 30 minutes prior to collecting your sample.


Collection

Tube A

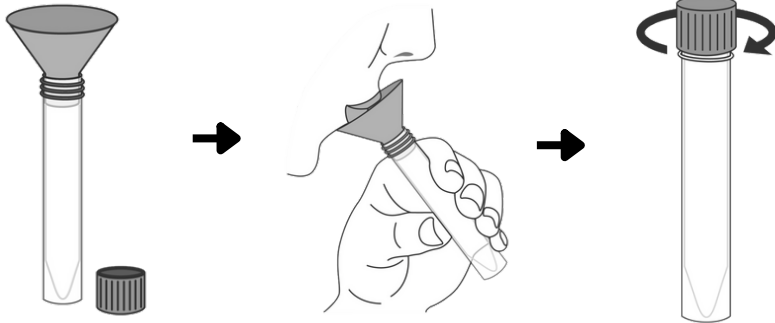


Collection

Tube B

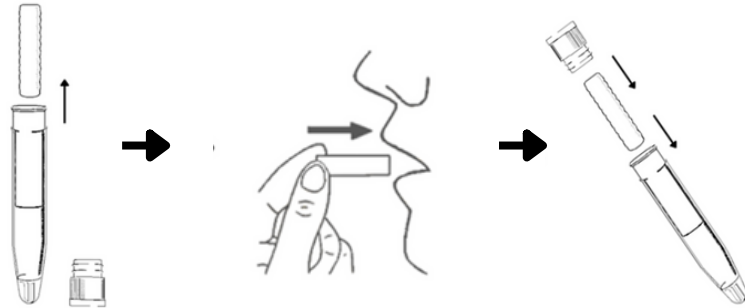

Package
& Ship

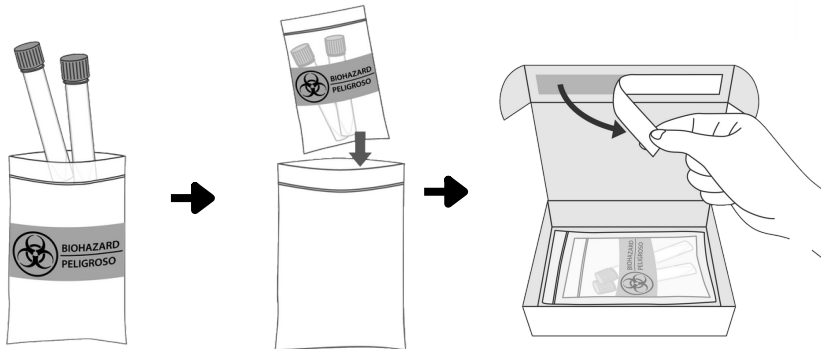

1. Print patient name, date of birth, and collection date on barcode labels attached to both collection tubes
2. With clean hands, place funnel in opening of saliva collection tube (A)
3. Spit liquid saliva into collection tube; collect between 1 and 2 mL as marked on the tube
4. Discard funnel and seal collection tube with cap
5. Thoroughly rinse mouth with water for 30 seconds
6. Remove cap from salivette collection tube (B); place salivette (cotton roll) in cheek and gently chew for 2 minutes (longer if little saliva is produced).
7. Remove salivette and place in salivette collection tube; secure cap
8. Seal both collection tubes inside biohazard bag; seal biohazard bag inside clear bag
9. Pack box with sealed collection tubes; remove adhesive liner; close and seal box; Ship via USPS

IMMYLabs must receive sample within 14 days of collection date.